



Technical Bulletin

Care & Cleaning Instructions for Nauga Soft Vinyl

Dear Doctor,

Your new Royal Dental patient chair is using Nauga Soft protected with BeautyGuard coating. Unlike standard vinyl, BeautyGuard protected Nauga Soft can withstand stronger cleaners and these instructions should only be used on BeautyGuard coated vinyl or damage may occur to the surface of other vinyl.

NOTE: Although Nauga Soft with BeautyGuard can resist staining much longer than some vinyl, a stain not cleaned right away has more potential to set and not be removable.

For light soiling:

- A solution of 10% household liquid dish soap with warm water applied with a soft damp cloth will remove most soiling.
- If necessary, a solution of liquid cleanser and water applied with a soft bristle brush. Wipe away the residue with a water-dampened cloth.

For Heavier soiling not solved by above method:

- Dampen a soft white cloth with lighter fluid (naphtha) and rub gently. Rinse with a water-dampened cloth.
- **Note:** Use **extreme** caution with this method. Complete only in a well-ventilated area and away from any open flame.
- **Note:** You should **try this method on an inconspicuous spot** before using it on the original stain/soiling.

Do not use bleach solutions on any Royal Dental products. Some materials and finishes can be damaged over time by contact with bleach so we discourage its use on our products.