



Royal Dental  
Manufacturing, Inc.  
12414 Highway 99  
Everett, WA 98204-5548  
Tel. (425) 743-0988  
Fax (425) 743-3588

## CARE AND CLEANING INSTRUCTIONS FOR ROYAL UPHOLSTERY

Dear Doctor,

All new Royal Dental chair upholstery selected from the Royal Standard Color Guide is protected with a special surface coating that resists stains and discolorations. \_

**Buildups of disinfectant are known to harm the upholstery's protective coating.**

**Using barriers to control infection in lieu of dental surface disinfectants will most effectively increase the longevity of the chair's upholstery.**

**If you must use disinfectants then clean the chair upholstery afterwards as described below.**

**NOTE: The longer a stain is allowed to set, the more difficult it will be to remove.**

### **For Daily Cleaning:**

- Use a solution of 10% household liquid dish soap mixed with warm water and apply with a soft damp cloth to remove most soiling and buildups. If necessary, rinse, re-dampen the cloth and continue working out remaining residue. Then wipe the surface clean with a soft clean water-dampened cloth.
- Formula 409 or Fantastik spray cleaners may also be used. Be sure to rinse with a soft clean water-dampened cloth afterwards. **Do not use Citrus, Bleach or other variations of these cleaners, only the original formulas.**

### **For more difficult stains not solved by above method:**

- Dampen a soft white cloth with a standard 70% formulation of Isopropyl (rubbing) Alcohol and rub an inconspicuous area of the material first, insure that no discoloration has occurred, and then rub the stained area gently 6 times. **Note: Use extreme caution with this method. Complete only in a well-ventilated area and away from any open flame.**
- Rinse with a water-dampened cloth.

**Do not use Acetone (finger nail polish remover), Sodium Hypochlorite (bleach), Naphtha (lighter fluid) or D-limonene (citrus cleaners). These are known to damage the upholstery finish.**