



CARE AND CLEANING INSTRUCTIONS FOR BOLTAFLEX VINYL

Your Royal Dental Domain patient chair is upholstered with Boltaflex vinyl protected with PreFixx coating. **It can resist damage caused by most dental surface disinfectants, providing the disinfectants, as well as perspiration stains, are removed daily with mild soap and water in order to prevent build up.** Unlike other vinyl, PreFixx protected Boltaflex is manufactured specifically to withstand damage caused by strong cleansers; therefore, **these cleaning instructions should only be used on PreFixx coated vinyl** or damage may occur to the surfaces of other vinyl. The following are guidelines for stain remove:

NOTE: Although Boltaflex with PreFixx can resist staining much longer than non PreFixx coated vinyl, a stain not cleaned right away has more potential to set and not be removable.

Kinds of Stains

Inks, ballpoint, markers, pencil, blood products, feces, urine, perspiration stains, medications, make-up, shoe polish, rubber heel marks.

Cleaning Method

Treat stain immediately, before it sets with household cleaners (like Fantastic or Formula 409) and water. Rinse thoroughly with clean water to remove suds or loosened dirt lodged in depressed surfaces. Scrub with generous amounts of rubbing alcohol-type cleaners. If stain remains, lightly dab with nail polish remover (50% acetone / 50% water). Rinse with water and wipe dry with clean cloth.