



Technical Bulletin

Care & Cleaning Instructions for Boltaflex® Vinyl

Dear Doctor,

Your new Royal Dental patient chair is upholstered using Boltaflex vinyl protected with PreFixx coating. ***It can resist damage caused by most dental surface disinfectants, provided the disinfectants, as well as perspiration stains, are removed daily with mild soap and water in order to prevent build up.*** Unlike other vinyl, PreFixx protected Boltaflex is manufactured specifically to withstand damage caused by strong cleaners. Therefore, ***these cleaning instructions should only be used on PreFixx coated vinyls*** or damage may occur to the surface of other vinyl. The following are guidelines for stain removal:

NOTE: Although Boltaflex with PreFixx can resist staining much longer than non-PreFixx coated vinyl, ***a stain not cleaned right away has more potential to set and not be removable.***

Kinds of Stains

Stain group 1: Ketchup, chocolate, oil, juice, urine, blood, coffee

Stain group 2: Eye shadow, crayon, grease

Stain group 3: Tobacco tar, felt marker, mustard, lipstick, ballpoint pen


Cleaning Methods

Method 1: Treat stain immediately, before it sets, with household cleaners (like Fantastik or Formula 409) using a soft cloth or sponge. Rinse thoroughly with clean water to remove suds or loosened dirt lodged in depressed surfaces.

Method 2: Rub with soft cloth and rubbing alcohol-type cleaner. Rinse with water and dry.

Method 3: Lightly dab with a mixture of 70% water and 30% nail polish remover. Rinse with water and wipe dry with clean cloth.

Do not use bleach solutions on any Royal Dental products. Some materials and finishes can be damaged over time by contact with bleach so we discourage its use on our products.

 12414 Highway 99 • Everett, WA 98204
Phone 425.743.0098 • Fax 425.743.3588

Doc# 6096B2 Boltaflex cleaning