



CHAIR OPERATING INSTRUCTIONS

Royal Dental Chairs

1. To operate chair back, press the top rocker switch on the side of the chair backrest. Press the bottom of the switch to recline the chair back, and the top of the switch to raise it.
2. To raise the chair base, press the top of the lower rocker switch. To lower the base, press the bottom of the rocker switch.
3. Auto Position (option) – Press the engraved side of the auto position switch, on position. The chair base will rise and the chair back will lower to its pre-set position. **NOTE:** *After Auto Position has been pressed, it must be turned off before the manual control rocker switches can be used.*
4. Auto Return – Press the engraved side (AR) of the auto return switch, on position. The chair back will rise and base will return to the lowest position. **NOTE:** *After Auto Return has been pressed it must be turned off before other rocker switches or the auto position option can be used.*
5. Four-Way Foot Control (Option) –
 - a. Press the inside of the left pedal to raise base.
 - b. Press the inside of the right pedal to recline chair back.
 - c. “a” and “b” may be used at the same time.
 - d. Press the outside of the left pedal to lower base.
 - e. Press the outside of the right pedal to raise chair back.**NOTE:** *Auto Position and Auto Return switches must be turned off for foot controls to work.*
6. Patient Entry or Exit – Grasp armrest at mid-point, lift and swing outward to middle lock stop. To return armrest, lift armrest at mid-point and swing to forward position. Entry or exit can be done from either side of the chair. **NOTE:** *For wheelchair patients, swing armrest past mid-point stop, toward the back of the chair to allow easier entry/exit.*
7. Chair Maintenance/Cleaning Instructions – See “Care and Cleaning Instructions.
8. Base Lubrication – No lubrication is required with plastic drive nut. **NOTE:** *If base shrouds are off for other reasons, grease slide rods.*