



**Royal GP2 Chair
PD2 Chair**

INSTALLATION INSTRUCTIONS

CHECK CARTON FOR SHIPPING DAMAGE. ALL FREIGHT CLAIMS SHOULD BE DIRECTED TO THE CARRIER.

UNPACKING CHAIR BASE:

1. Remove six lag bolts from cover. Remove cover. Check for damage.
2. Remove two lag bolts from each side brace. Remove side braces.
3. Remove four bolts holding base to box.
4. Lift base from carton and place it in location where chair is to be used.

UNPACKING CHAIR TOP:

1. Cut bands and remove carton cover. Check for damage.
2. Remove seat cushion and bolts holding chair top to shipping pad.
3. Remove plastic bag containing four bolts and four washers from seat cushion center cross support.

ASSEMBLING TOP TO BASE:

1. Using two people, lift chair top out of carton and place on base, aligning the four holes.
2. Using the four bolts and washers that were removed from the seat cushion, bolt upper to base, tightening all four bolts securely.
3. Connect the Molex connectors between the base and upper, then plug the power cord into the power receptacle.
4. Straddle the seat cushion retainers, (rear of cushion), over the chair top frame rails. Move cushion toward rear of chair until holes in cushion retainers align with the holes in chair plate. Install one 3/8-16 x 1 _ bolt through chair plate into each cushion retainer and tighten securely.

TESTING CHAIR:

1. Operate the top rocker switch on each side by pressing bottom side of switch to lower the chair back, and upper side to raise. Leave the chair back partly reclined.
2. Press upper side of lower rocker switch to raise base, and bottom side to lower. Leave base partly raised.
3. Press automatic return button. Chair back should raise and base should go to the lowest position.

NOTE: After Auto Return has been pressed, it must be turned off before other rocker switches can be used.