

Royal Signet 2230 Operating / Programming Instructions

Chair Backrest

To operate the chair backrest, locate the top rocker switch on the backrest control panel. Press the bottom of the switch to recline the backrest, press the top of the switch to return the backrest upright.

Note: *The seat cushion is synchronized to move with the backrest.*



Chair Seat Height

To operate the chair height, located the middle rocker switch on the backrest control panel. Press the bottom of the switch to lower the chair, press the top of the switch to raise the chair.

Auto Return (Exit Position)

To lower the chair completely and position backrest upright, locate the bottom rocker switch. Press and the bottom side of the switch “two lines”.

Auto Position

To raise the chair and recline the backrest to its pre-set position, locate the bottom switch on the backrest control panel. Press the topside of the switch and release. Next, press and release the desired manual control switch. This actuates the chair.

Programming Auto Position


Run chair to desired position using manual controls. Push and hold AP. While holding AP, push and hold the preferred manual control switch until you hear 2 beeps (approx. 5 sec.).



Patient Entry or Exit - Armrest

Grasp the armrest in the middle, lift and swing outward to the middle lock stop. To return the armrest, lift armrest at in the middle and swing to forward position. Both armrests swing to allow for left and right chair entry or exit. **Note:** **For maximum access, swing armrest past mid point stop, toward the back of chair to allow easier entry or exit.**

Manual Switches

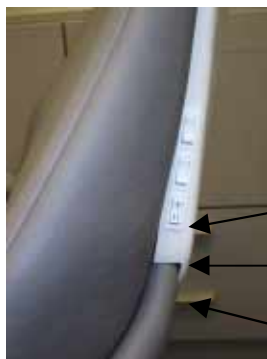

Preset 1
Preset 2
Auto Position Switch