

# Wall & Cabinet Dental Light

## Preinstallation

Carefully unpack the light. Inspect all of the items received to assure that there has been no shipping damage. If damage is discovered, contact the carrier and report and file the appropriate claims with the carrier within 24 hours after deliver.

## Tools Required

Allen wrenches, socket set, bubble level, tape measure, screwdrivers, hammer and drill.

## Installation

1. Install the wall and cabinet plate with the bushing at the top. Assure that the plate is level front to back and aside to side by positioning the bubble level as shown below.
2. Insert the light bushing into the collar and lubricate the bushing.
3. Install the arm into the bushing. It is recommended that one person feed the low voltage power cable through the bushing, while an assistant holds the arm/head assembly, and after the cable is in place, lower the arm hub into the bushing.
4. The transformer can now be positioned as desired for cabinet only.
5. Plug the low voltage cable connector into the receptacle in the transformer cover..
6. Plug the hospital grade plug from the transformer into a 120V hospital grade receptacle cabinet mount only. Consult Electrician for wall mount.
7. Turn the light on by moving the toggle to the far left. The switch is immediately behind the light head. The same switch is utilized to change the light to maximum brilliance for color matching (5000 K) by moving the switch to the far right position.

